DUAL-CURE CORE COMPOSITE

REF S300, Various

Rx Only | **EN**

For Safety Data Sheet (SDS) go to www.parkell.com

INSTRUCTIONS FOR USE

parkell[®]

A00361revM0320

DESCRIPTION:

Parkell's Absolute Dentin[®] is a strong, dual-cure, composite resin material that is highly filled with fluoridated barium/aluminum glass for radiopacity. It is intended for core fabrication, and is supplied in two convenient packages: a 50 ml/110 gm cartridge and a 10 ml/22 gm cartridge, each to be expressed with the appropriate automix gun. The uncured material flows well and stacks without slumping, while the cured material "feels" much like dentin when prepared with rotary instruments.

Absolute Dentin works with all dentin/enamel bonding agents that are compatible with dual-cure or self-cure composites, including all of Parkell's 4-META-based bonding products (Brush&Bond[®], Amalgambond[™], C&B-Metabond[®] and SEcure[®]). It is also compatible with Parkell's Add&Bond[™], DuraFinish[™] and DuraFinish[™] All-Cure.

Absolute Dentin kits are available in three shades: Tooth Shade (similar to A2/A3), Arctic White (A1/B1), and Blue (easily distinguished from surrounding tooth structure). Each kit contains an appropriate number of mixing tips and intraoral tips.

DIRECTIONS:

1. Apply the appropriate dentin/enamel bonding agent to the prepared tooth and cure it following the material's instructions. Since Absolute Dentin is a dual-cure material, make sure that the bonding agent chosen is compatible with DUAL-CURE or SELF-CURE composites. Certain non-Parkell self-etching bonding agents may interfere with the setting ability of dual-cure composites, so check the instructions for your bonding agent before proceeding.
2. Setting up the product for use:
 - a. If using the 50 ml automix cartridge, insert it into any DS-50 (1:1 / 2:1) impression gun such as Parkell's REF S343. Remove and discard the blue sealing cap or used mixing tip from the cartridge, and express out a small amount of material through both round orifices. Then attach a yellow mixing tip to the cartridge, and a black intraoral tip to the mixing tip.
 - b. If using the 10 ml automix cartridge, insert it into the appropriate 10 ml (1:1) impression gun such as Parkell's REF S327. Remove and discard the brown sealing cap or used mixing tip from the cartridge, and express out a small amount of material through both round orifices. Then attach a brown mixing tip to the cartridge, and a clear, angled intraoral tip to the mixing tip.

3. Express out a pea-sized amount of Absolute Dentin through the tip onto a pad to ensure a complete mix of base and catalyst pastes. Immediately express the mixed material directly from the cartridge onto the bonding agent coated tooth. A matrix band or core-forming capsule may be used to contain the material.
4. Absolute Dentin will self-cure without light in about 4 minutes. The light initiator in the product is camphorquinone (CQ), which has an activation range of between 460 and 480nm. Absolute Dentin will light-cure to a depth of 3mm with a 40 second exposure to any dental curing light producing an intensity of at least 400 mW/cm². Actual clinical depth of cure will depend on factors such as time, light intensity, device condition, distance between the light and the restoration, and material shade.
5. Once fully cured, prepare the core as usual with rotary instruments of choice.
6. Leave the used mixing tip in place on the cartridge to serve as the sealing cap between uses. The cartridge, used mixing tip and gun may be disinfected with appropriate dental surface disinfectants. Check with the disinfectant manufacturer before use.

STORAGE AND USE:

Store the material in a cool, dry place. For longest shelf life, refrigeration without freezing is preferred. After removal from the fridge, allow Absolute Dentin to warm to room temperature for best handling. Cold material may be difficult to express and take longer to cure.

CONTRAINDICATIONS:

Absolute Dentin should not be used on or by persons who are methacrylate sensitive. Uncured material should not be swallowed.

TIPS FOR DENTAL PRACTITIONERS

The following information should help dental practitioners reduce the risk of cross-contamination between patients when using "disposable multiple-use dental dispensers" (referred to as "these items" below).

- Apply disposable barrier sleeves/wraps over these items before use with each patient.
- Use new, uncontaminated gloves when handling these items.

- Utilize dental assistants to dispense material from these items for the dentist.
- Avoid contact between these items and the patient's orofacial tissues.
- Do not reuse or reprocess these items with chemical wipes or disinfectants if they become contaminated.
- Do not attempt to sterilize these items, or immerse them in high-level chemical disinfectants, as this may damage them or the material contained within them.

WARRANTY AND TERMS OF USE:

For full Warranty and Terms of Use information, please see www.parkell.com. Safety Data Sheets (SDS) are available at www.parkell.com. Parkell's Quality System is certified to ISO 13485.

AVAILABLE KITS AND REFILLS:

Absolute Dentin 50 ml Kits:

Includes (1) 50 ml split cartridge (110 gm / paste total) of one shade, (40) mixing tips and (40) intraoral tips.

- (REF S300) **Arctic White Shade**
- (REF S301) **Tooth Shade**
- (REF S307) **Blue Shade**

Absolute Dentin 50 ml Accessories:

- (REF S302) **Yellow Mixing Tip Refill for 50 ml** (30)
- (REF S308) **Dark Grey Intraoral Tip Refill** (50) - use with (REF S302)
- (REF S343) **1:1 / 2:1 Dispensing Gun for 50 ml split cartridge**

Absolute Dentin 10 ml Kits:

Includes (2) 10 ml cartridges (20 ml - 44 gm / paste total) of one shade, (16) mixing tips and (16) intraoral tips.

- (REF S304) **Arctic White Shade**
- (REF S305) **Tooth Shade**

Absolute Dentin 10 ml Accessories:

- (REF S328) **Brown Mixing Tip Refill for 10 ml** (30)
- (REF S335) **White Intraoral / Root Canal Tip Refill** (50) - use with (REF S328)
- (REF S329) **White Intraoral Tip Refill** (50) - use with (REF S328)
- (REF S327) **1:1 Dispensing Gun for 10 ml cartridge**

EXPLANATION OF SYMBOLS USED

	Professional use only
	Follow instructions for use
	Temperature limit
	Keep away from sunlight
	Use-by date (expiration)
	Batch code
	Catalogue / stock number
	Package contents
	Do not use if package is damaged
	Manufacturer
	Single use only.
	10 ml Cartridge
	50 ml Split Cartridge
	Mixing Tip
	Intraoral Tip
	Intraoral / Root Canal Tip

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**Directa AB**, Finvids väg 8,