

Its-a-Wrap™ Comfort Barriers for Digital X-Rays

Large | Yellow **REF** BWRAPL / Small | White **REF** BWRAPS


INSTRUCTIONS FOR USE

- **Large | Yellow: Adjustable to fit all size 2 sensors**
- **Small | White: Adjustable to fit all size 0-1 sensors**


Photos used in demonstration are Its-a-Wrap™ Comfort Barriers Large, Yellow.



1
Insert sensor image-side down. Remove “Peel & Wrap” liner and fold over for a snug fit.

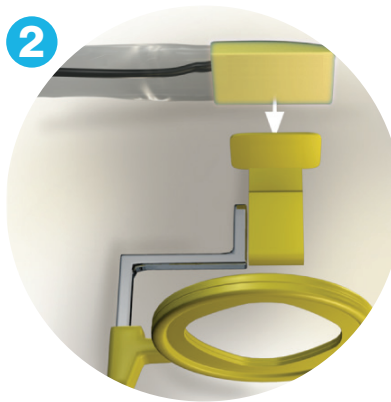

2
Attach to Schick adhesive-backed CDR® Holder Tab (yellow for posterior periapicals, white for anterior periapicals and blue for bite-wing X-Rays).

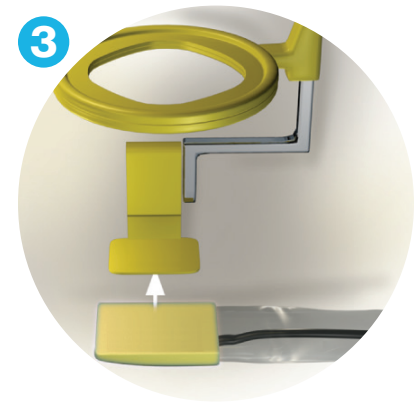

3
For posterior periapicals, take images of first two quadrants, remove CDR Holder Tab adhesive-backed bite tab. Adjust holder tab aiming arm and ring. Reattach bite tab and take X-Rays of two remaining quadrants. Discard after use.



Crosstex International, Inc., a Cantel Medical Company
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Not made with natural rubber latex.

Made in China

BWRAP-IFU-EN / REV B - 04/2021

CROSSTEX is a trademark of Crosstex International, Inc., its affiliates or related companies. All other product and company names referenced are trademarks of their respective owner.

©2021 Crosstex International, Inc. All rights reserved.