

EmPower™

3-in-1 Multi Enzymatic Detergent




Instructions for use in Dental Clinics:

Use fresh EmPower for each instrument or set of instruments.


Pretreatment/Presoak:

- For pretreatment, spray Empower Foam evenly and directly onto instruments/equipment directly after use to ensure soils are completely coated. Re-apply as needed to keep instruments moist.
- Rinse instruments/equipment before further processing.
- For presoak, add 1 oz. (1 pump yields 1 fl. oz.) of concentrate to one gallon of warm water (68°F-104°F).
- Soak instruments and equipment immediately after use, until soil is dissolved and removed.
- Soak for a minimum of 1 minute. Soak longer if matter is dried on, until visibly clean.
- Immerse instruments completely and aspirate solution into any channels and lumens.
- If using Ultrasonic cleaning, follow steps a-c below:

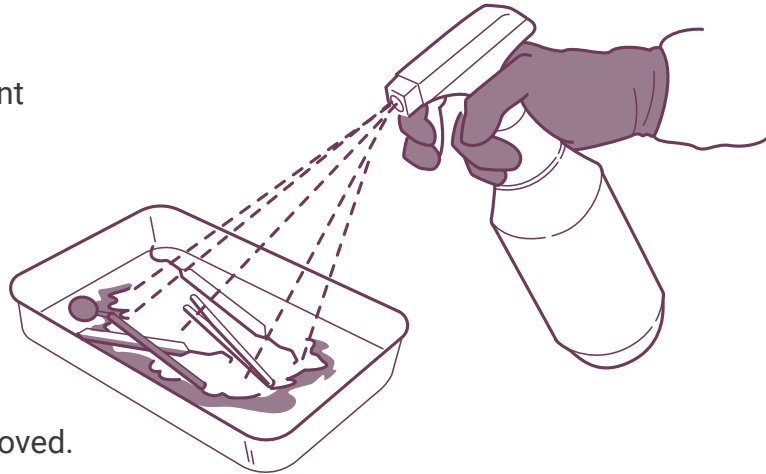

Ultrasonic Units:

- Add 1fl. oz. of concentrate per gallon of water. Recommended water temperature is 68°F-104°F.
 - A soak time of 2 minutes or more is recommended for ultrasonic cleaners.
 - Process instruments according to manufacturer's directions.
- Use an instrument brush for light mechanical cleaning if necessary (Refer to the instrument manufacturer's instructions for cleaning).
 - Following soak period, visually inspect for cleanliness and rinse thoroughly with warm water.
 - Let Dry. After cleaning with EmPower, equipment and instruments must be disinfected/sterilized according to appropriate procedure.

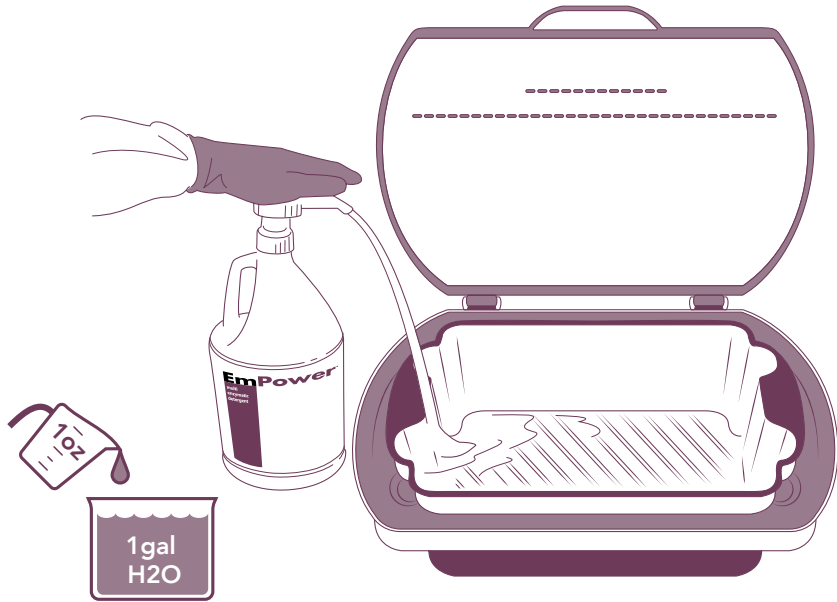

Dental Evacuation System Cleaner:


- Add 1fl. oz. of EmPower concentrate to the dispenser bottle and fill the bottle with warm water to the 1 gallon mark.
- Attach diluted solution container to the appropriate saliva ejector and/or high-volume evacuation line.
- Open evacuation line valve(s) and suction off 1-2 quarts per operator.
- Repeat between patients or as necessary.

Ordering Information

Part #	Description
10-4100	EmPower Multi-Enzymatic Detergent 1gal
10-4400	EmPower Multi-Enzymatic Fragrance Free 1gal
10-4150	EmPower Multi-Enzymatic Fragrance Free 5gal
10-4224	EmPower Foam (Dual-Enzymatic) 24oz Spray

Accessories

46-1001	EmPower Dispensing Pump - 1 Gallon
10-4199	EmPower Dispenser

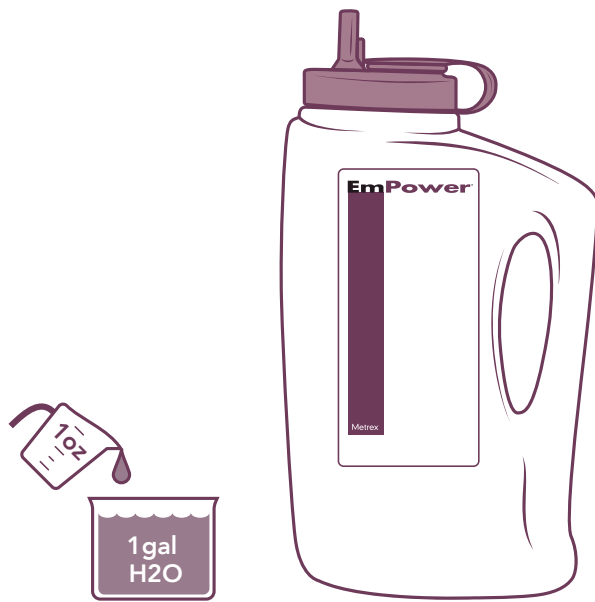

1. Refer to Empower Product Labels and SDS for detailed Use Directions and Precautionary/ First Aid Statements.
2. Refer to Device Manufacturer Directions for compatibility information.
3. Empower and Empower Foam are not disinfectant/ sterilant.
4. Utilize proper PPE according to CDC recommendations.
Refer to Device Manufacturer Directions for any applicable terminal sterilization/ disinfection procedures.

Scan the QR Code
for more resources on EmPower
kerrdental.com/kerr-totalcare

