

Foilocote

Liquid Foil Substitute Instructions

Foilocote is designed as a tin foil substitute and should be used as a separating medium and model seal prior to packing acrylic.

Using a brush, paint the entire surface of the model with Foilocote, preferably when the model is **not** hot or warm. A single coat is usually sufficient for separation; however, a second coat may be applied after a few minutes (five minute wait minimum) or anytime later, and give excellent results.

Painting the entire surface of the model facilitates removal of excess acrylic in trial packing.

When Cellophane **is not** used in curing please be sure that the model side has had two coats of Foilocote. This application may be done after the trial packing procedure.

Foilocote may be removed from surfaces with water. Please see SDS (MSDS) for further information.

REV: 9/30/14



361 Farmington Ave • Louisville, KY 40209 • 800.626.5651 • www.whipmix.com