

Core buildup in 60 seconds or less with SuperCure®.

*The easiest way to speed your
core buildups.*



With SuperCure and SuperCure-Q, no mixing or automix nozzles are required. Simply inject directly into the prep and build a high strength core in 60 seconds or less.

SuperCure and SuperCure-Q are light-activated core buildup materials packaged in single-dose pre-filled tips. The non-slumping formula eliminates the need to use a matrix and the translucent material cures deep with any light source.

- Fast light-activation and high depth of cure – 8.9mm in 40 seconds
- Single-dose, pre-filled syringe tips for maximum asepsis
- Non-slumping formula so there is no need to use a matrix
- No automix nozzles
- Available in two shades – natural and contrast
- Radiopaque



Ordering is easy. Call 800-235-5862 or contact your preferred dealer.
Visit centrixdental.com for more information.



Original SuperCure®
Cuts like dentin.



SuperCure®-Q
Cuts harder than dentin.

CENTRIX
SuperCure®
CENTRIX
SuperCure®-Q

Light-Activated, Deep Curing
Core Buildup Material



centrix®
Making Dentistry Easier.™

CENTRIX
SuperCure®

CENTRIX
SuperCure®-Q

Light-Activated, Deep Curing
Core Buildup Material

	Original SuperCure	SuperCure-Q
Compressive strength	235 Mpa	235 Mpa
Flexural strength	87 Mpa	104 Mpa
Diametral tensile strength	50 Mpa	50 Mpa
Filler type	Barium borosilicate glass	Barium borosilicate and quartz glass
Cure depth, (ADA), 40 seconds	8.9mm	8.9mm

6-STEP GUIDE TO BUILDING A VOID-FREE, BONDED, COMPOSITE POST & CORE



1. Prepare the tooth. Bevel the edges to eliminate sharp angles.



2. Etch the post hole, rinse thoroughly and dry.



3. Apply bonding agent according to manufacturers instructions



4. Insert prefilled SuperCure tip to the bottom of the prep and inject a 2mm layer of material. Cure with light for 20 seconds.



5. Continue building up the core by injecting and slowly withdrawing the tip.



6. Light activate for 40 seconds and then cut and shape the core. The result is a solid, securely bonded post and core.

ORDERING INFORMATION:

ORIGINAL SUPERCURE

Original formula of barium borosilicate glass filler cuts like dentin.

.50 GRAM TIPS

REF 310141 30 x .50g tips, natural shade
REF 310143 30 x .50g tips, contrast shade

.25 GRAM TIPS

REF 310140 30 x .25g tips, natural shade
REF 310142 30 x .25g tips, contrast shade

SUPERCURE-Q

Reinforced formula with quartz glass filler, cuts harder than dentin.

.50 GRAM TIPS

REF 310146 30 x .50g tips, natural shade
REF 310147 30 x .50g tips, contrast shade

SUGGESTED ACCESSORY

REF 130700 E/Z™ Syringe
(Steel barrel end)



Ordering is easy.

Call 1-800-235-5862
or contact your preferred dealer.

centrixdental.com



Our Guarantee — If you are not 100% satisfied, simply return the product for a complete refund. It's that easy.



centrix®
Making Dentistry Easier.™