

Vella®: The Versatile Varnish.



The fluoride varnish experience that's actually pleasant for both you and your patients.

No matter their age, patients are happier when you apply Vella's smoother, less gritty formula with a "thin to win" technique, leaving them with just the sweet scent and taste of fresh, fun flavors.

You're happier too. Vella's clear formula applies easily in a wet field, dries quickly, and provides fast fluoride release and uptake at two hours¹. Sweetened with Xylitol, Vella contains NuFluor®, our combination chemistry featuring fluoride, calcium and phosphate for greater relief of hypersensitivity.

Gluten-free Vella is available in two package options: The sturdy Vella stick, or VellaMini for the back-of-the-glove application technique. Use it and make everyone happy.

VELLA®

5% Sodium Fluoride Varnish with NuFluor®



Absolute
Satisfaction
Guaranteed.

¹Data on file.



Strawberry



Melon



Spearmint



Bubble Gum



Caramel

PREVENTECH®







4330-C Matthews-Indian Trail Road • Indian Trail, NC 28079 • 800.474.8681 • 704.849.2416 • fax: 704.849.2417 • preventech.com



Same Excellent Varnish. Two Excellent Delivery Systems.

VELLA®






5% Sodium Fluoride Varnish with NuFluor®

Item No.	Quantity	Flavor
 770013	Box of 35 (.5mL)	Spear-mint
770033	Box of 100 (.5mL)	Spear-mint
 770023	Box of 35 (.5mL)	Strawberry
770043	Box of 100 (.5mL)	Strawberry
 770073	Box of 35 (.5mL)	Melon
770083	Box of 100 (.5mL)	Melon
 770103	Box of 35 (.5mL)	Bubble Gum
770113	Box of 100 (.5mL)	Bubble Gum
 770143	Box of 35 (.5mL)	Caramel
770153	Box of 100 (.5mL)	Caramel



VELLAMINI®

5% Sodium Fluoride Varnish with NuFluor®

Item No.	Quantity	Flavor
 772213	Box of 35 (.5mL)	Spear-mint
772233	Box of 100 (.5mL)	Spear-mint
 772223	Box of 35 (.5mL)	Strawberry
772243	Box of 100 (.5mL)	Strawberry
 772273	Box of 35 (.5mL)	Melon
772283	Box of 100 (.5mL)	Melon
 772103	Box of 35 (.5mL)	Bubble Gum
772113	Box of 100 (.5mL)	Bubble Gum
 772143	Box of 35 (.5mL)	Caramel
772153	Box of 100 (.5mL)	Caramel




VELLAMini: The Glove Technique Improves the Application Process




- 1 Hold and knead the VELLAMini pack to ensure uniform viscosity. Holding top section, gently tear open pack at perforation. Discard top section.
- 2 Dispense VELLA varnish directly on back of glove.
- 3 Using UltraBrush 2.0, gently mix and pick up thin layer of varnish to apply for an improved patient experience. Begin with mandible to ensure moist (but no pooling) tooth surface. Use free hand for retraction and easier access to all tooth surfaces.

Superior Uptake into Treated Enamel

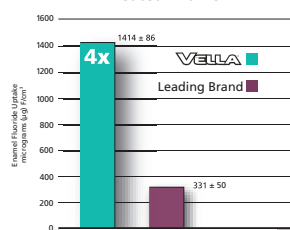

Superior Fluoride Release through 12 Hours

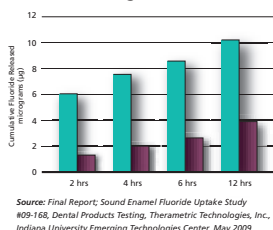

Source: Final Report, Sound Enamel Fluoride Uptake Study #09-168, Dental Products Testing, TheraMetric Technologies, Inc., Indiana University Emerging Technologies Center, May 2009.

- Smoother, less sticky, clear formula sets quickly in contact with saliva.
- Contains NuFluor, our patented combination chemistry containing calcium, phosphate and fluoride.
- Contains Xylitol and is gluten-free.
- Great fluoride uptake at 2 hours.¹
- Contains 22,600 ppm fluoride.
- Provides relief from sensitivity.
- n-Hexane free...Google it!

Buy 100, Get 35 FREE!
Or Bulk Special: Buy 10 Boxes, Get 5 Boxes FREE!

Name of Practice _____
 Address _____
 City _____ State _____ Zip _____
 e-mail _____

Just place your order for 1 or more 100-count boxes of VELLA or VELLAMini through the dealer of your choice. Then attach your invoice to this coupon and mail it to Preventech, so we can process your FREE Vella products. Allow 4 to 6 weeks for delivery. Offer expires 12/31/20.

- Strawberry
- Spearmint
- Melon
- Bubble Gum
- Caramel

☐ VELLA
☐ VELLAMINI

There's no limit to the number of boxes an office may order, but at least one box (100) must be purchased on a single invoice. Mixing Vella flavors and delivery systems is allowed. This offer may not be combined with any other offers.

Your satisfaction is 100% guaranteed. If any Preventech product does not live up to your standards, return it for a full refund. Customer Service: 1-800-474-8681

¹Data on file.

