


THE ORIGINAL SDF SYSTEM



NEW GENERATION OF DESENSITIZER WHICH IMMEDIATELY REMOVES TOOTH SENSITIVITY

A STAR PRODUCT CONTAINING THREE POWERFUL COMPONENTS:

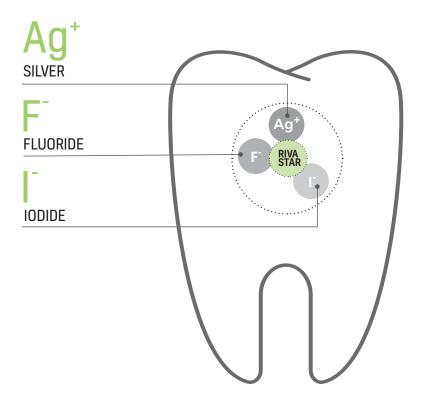

ASSISTS IN THE ELIMINATION OF BIOFILM¹⁻³ INHIBITS BIOFILM FORMATION¹⁻³ MAY ASSIST IN THE ALLEVIATION OF CARIES⁴

REFERENCES

- Knight GM, McIntyre JM, Craig G, Zilm PS, Gully N.; Inability to form a biofilm of Streptococcus mutans on silver fluoride-and potassium iodide-treated demineralized dentin. Quintessence international (Berlin, Germany: 1985) 2009;40:155.
- Knight GM.; The Pharmacological management of dentine to protect against plaque microorganism degradation. PhD Thesis University of Adelaide 2008.
- Knight GM, McIntyre JM, Craig G, Zilm PS, Gully NJ.; Differences between normal and demineralized dentine pretreated with silver fluoride and potassium iodide after an in vitro challenge by Streptococcus mutans. Australian Dental Journa I 2007;52:16-21.
- 4. Knight GM.; Silver Bullet. Australian Dental Journal 2010; Aesthetic update.

TOOTH DESENSITIZING AGENT

TWO YEARS AND BEYOND DESENSITIZING EFFECT

Clinical studies support the immediate effect of Riva Star and its long term effect of over two years.

INCREASING BOND STRENGTHS OF GICS

Pre-treating tooth surfaces with Riva Star is a great way to enhance bonding of glass ionomers to dentin.

PATENTED PROCEDURE

Unlike other silver fluoride systems, Riva Star's two step patented procedure minimizes the risk of staining. By applying the potassium iodine solution over the silver fluoride solution, a silver iodide precipitate is formed.

KEY FEATURES

Clinically tested and proven for decades

Esthetically pleasing for the patient - reduced staining

Immediate dentinal hypersensitivity relief for adults over the age of 21

Non- invasive - suitable for MID

Patented Technology

RIVA STAR - NO STAINING

Desensitizing treatment on the cervical region of the canine.



Before treatment



After 7 days of treatment

Desensitizing treatment on the cervical regions of premolars.




Before treatment



After 7 days of treatment

INSTRUCTIONS CAPSULES KIT

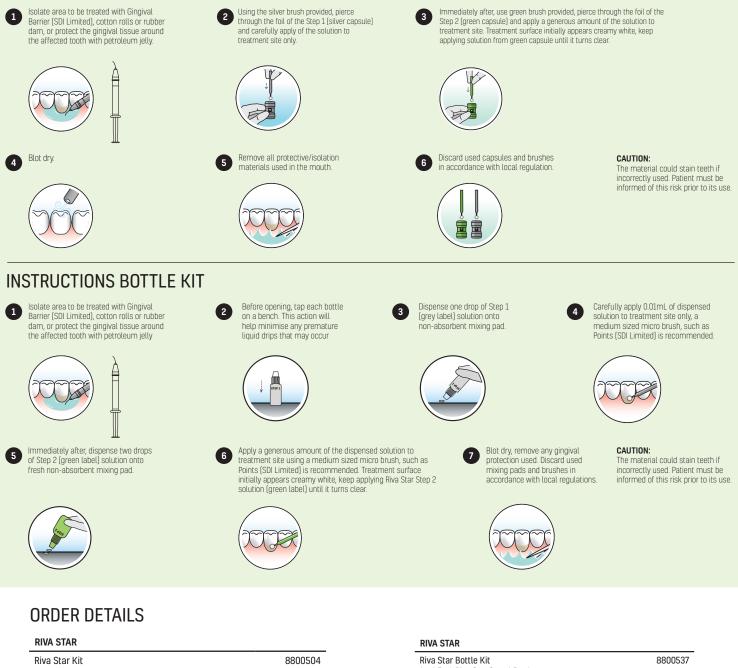

10 x Riva Star Step 1 Silver Capsules 10 x Riva Star Step 2 Green Capsules Accessories Riva Star Bottle Kit 1 x 1.5mL Riva Star Step 1 Bottle 1 x 3.0mL Riva Star Step 2 Bottle

> 11-2021 MILOUA29 D 0297 +D036M1004291/

ITALY 00800 0225 5734 NEW ZEALAND 0800 808 855 SPAIN 00800 0225 5734 UNITED KINGDOM 00800 0225 5734 USA & CANADA 1 800 228 5166



MADE IN AUSTRALIA by SDI Limited Bayswater, Victoria 3153 Australia www.sdi.com.au www.rivastar.com

AUSTRALIA 1800 337 003 AUSTRIA 00800 0225 5734 BRAZIL 0800 770 1735 FRANCE 00800 0225 5734 GERMANY 0800 100 5759